



Gli specialisti della vista in Farmacia

## PRE-ASSEMBLED GLASSES

### Warnings and precautions for use.

In order to properly assess the extent and nature of a visual defect, a prior eye exam is necessary.

In any case, assessment of the visual defect cannot be based on self-diagnosis or on the choice of glasses considered suitable by successive approximations. Pre-assembled glasses for presbyopia, industrially manufactured with lenses of identical dioptric power, are indicated EXCLUSIVELY for the correction of "common" presbyopia.

Pre-assembled glasses are suitable for near vision only. They are not to be used for driving, distance vision, or eye protection. Improper use or the use of a wrong lens power of glasses can cause various vision disorders, with possible headache. In this case an ophthalmological check is necessary. Only the ophthalmologist is able to assess the state of health of the eyes and the extent of visual disturbances.

Pre-assembled glasses for presbyopia should not be used in cases of "complex" presbyopia.

Pre-assembled glasses for presbyopia do not improve vision in cases of eye conditions, such as cataracts or macular degeneration. Presbyopia is a physiological and progressive weakening of vision which generally manifests itself after the age of 45 with a series of disturbances such as difficulty reading the newspaper or distinguishing small objects up close. Presbyopia is considered "common" when for the correction of the visual defect, both eyes need the same number of corrective dioptres (dioptres are the unit of measurement of visual capacity). Presbyopia is considered "complex" if there is astigmatism or anisometropia or when the number of dioptres in each eye is different.

Astigmatism is a vision defect due to an irregular curvature of the cornea which causes a blurred and distorted vision of images. Anisometropia is characterized by a different refractive error in the two eyes.

### Warnings

The pre-assembled glasses comply with Regulation (EU) 2017/745.

### Storage and Cleaning

"When not in use and for transport, keep the frame in its original case at a temperature between -10°C and +35°C"; Avoid exposure to the sun in unconditioned closed environments (e.g. car dashboard) because the temperatures reached in these conditions can damage the functionality of the frame.

In cases of occasional contact with agents such as salt water (sea water), swimming pool water, chemicals such as cosmetics, hairspray, sunscreens and insect repellents, immediate removal with a soft damp cloth is recommended. In the event of wear (e.g. scratches, dullness) original replacement spare parts are recommended. For cleaning, use a damp cloth and a normal neutral detergent and dry with a clean soft cloth. Avoid the use of solvents (e.g. alcohol and acetone) and aggressive chemical detergents which can compromise the functionality of the glasses.

MD

Utilissimi reading glasses are a **CLASS I MEDICAL DEVICE** Product compliant with the **UNI EN 14139:2010** standard, **Regulation (EU) 2017/745** and **Ministerial Decree 26/11/2004**

The model code and lot number are printed on the inner side of the temple, preceded by the LOT symbol (indicates week/year)

LOT



**NOT SUITABLE FOR  
ROAD DRIVING**

### Precautions for use:

- Only for near vision and reading
- Only regular eye examinations by a professional ophthalmologist can determine visual needs and eye health
- Not intended for driving or operating a vehicle
- Not intended for distance vision
- Not intended for eye protection

### USER GUIDE FOR PRE-ASSEMBLED GLASSES

Dioptic power: shown on the glasses Interpupillary distance 62mm +/- 1mm



**Aurigane S.r.l.**  
Via dell'Economia, 32  
36100 Vicenza (VI) - Italy  
[www.utilissimi.net](http://www.utilissimi.net)

utilissimi

Gli specialisti della vista in Farmacia